

Instructions to use Voyager Web on Mac


Please follow the steps below to login to Voyager using web:

1 – Open Safari and type the URL

<https://patients.nowraradiology.com.au/web3> and then click Enter.

2 – Click on the link ‘mac users please click here’, just below Nowra Radiology’s Logo.

3 – Once the page loads click on ‘Voyager Imaging Web Loader’. The installer starts downloaded from Nowra Radiology’s Voyager web Server.

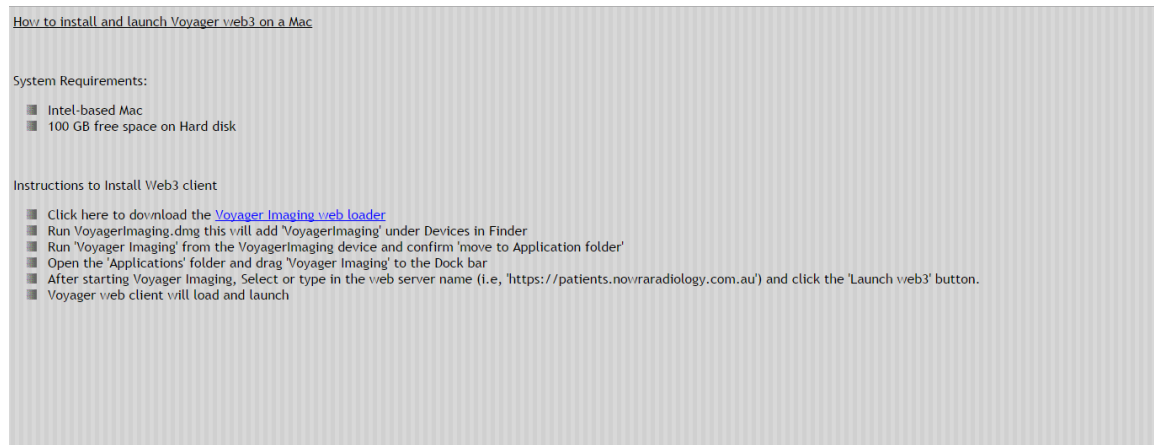


4 – Once the installer is downloaded, double click on ‘VoyagerImaging.dmg.zip’ via ‘Downloads’. This will add Voyager Imaging under Devices.

5 – Run Voyager Imaging and confirm to ‘more to application folder’.

6 – Open the ‘Applications’ folder and drag ‘Voyager Imaging’ to the Dock Bar.


7 – Go to ‘Dock Bar’ and open ‘Voyager Imaging’ .

8 – Voyager Web Launcher application will launch. Enter the Server Name as – <https://patients.nowraradiology.com.au> and then click on ‘Launch Web3’.

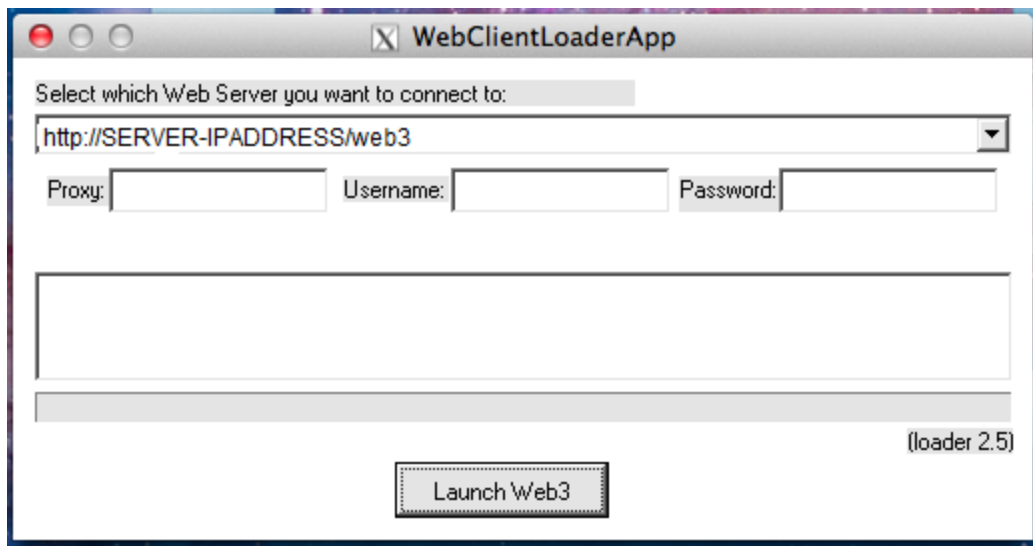

9 – Once Voyager Web loads up, put in your Username and Password and hit ‘Login’.

10 – Voyager Web will log into the Server. Click on the relevant checkboxes on left or go to ‘Quick Search’ on the top to find your patient’s images.

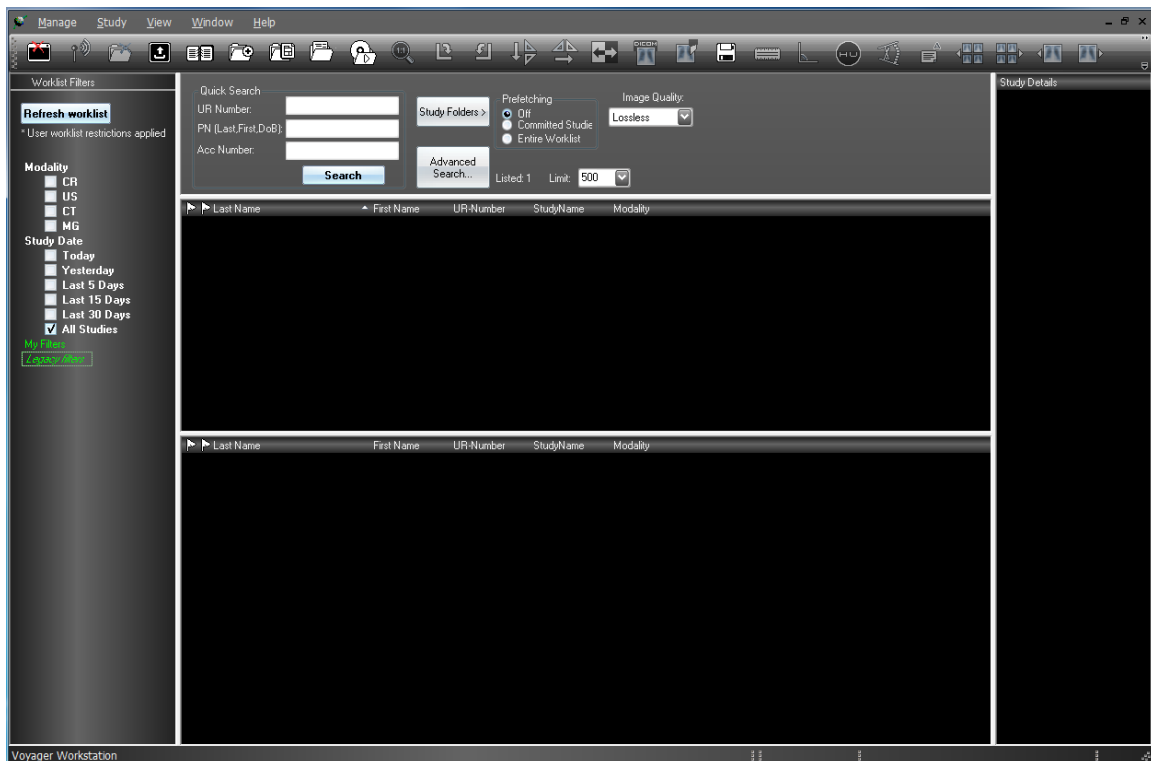

-END-